<b>DE</b>	<b>Larynx-Tubus LT-D</b>	Gebrauchsanweisung Seite 2-5
<b>GB</b>	<b>Laryngeal Tube LT-D</b>	Instructions for use Page 6-9
<b>FR</b>	<b>Tube Laryngé LT-D</b>	Manuel d'utilisation Page 10-13
<b>ES</b>	<b>Tubo Laríngeo LT-D</b>	Instrucciones de uso Página 14-17
<b>IT</b>	<b>Tubo Laringeo LT-D</b>	Istruzioni d'uso Pagina 18-21

**STERILE EO**

**CE 0123**

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Einsteinstrasse 1 • DE-72172 Sulz a.N. • Germany  
 Tel: +49 7454 / 95 96 0 • Fax: +49 7454 / 95 96 33  
 e-mail: info@vbm-medical.de • www.vbm-medical.de

# VBM Larynx-Tubus LT-D

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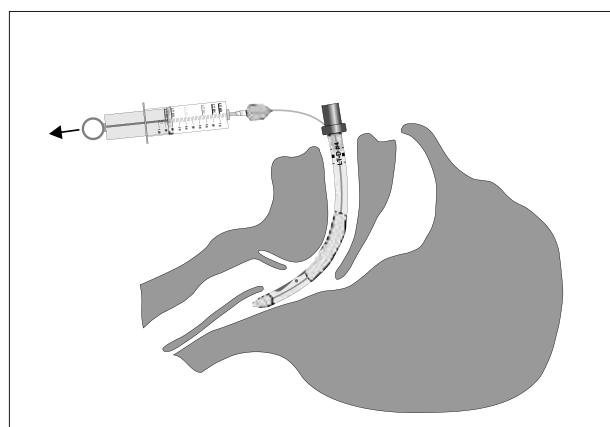
- latex free
- for single use

## ISO Connector

- color coded, for immediate identification of different sizes

## Inflation Line

- single lumen for inflation and deflation of both cuffs

## Teeth Marks

- for orientation during insertion and repositioning

## Proximal Cuff

- stabilizes the tube and blocks oro- and nasopharynx

## LT-D Cuffs

- are high volume cuffs which adjust ideal to the anatomical situation

## Distal Cuff

- blocks esophageal inlet
- reduces possibility of gastric ventilation

## Ventilation Holes

- lie in front of the larynx
- allow suctioning and the passage of a fiberoptic bronchoscope

**Box of 10**, complete with 10 LT-D, 1 color coded syringe and 1 biteblock

Size	Patient	Patient Size	Connector Color	REF
2	Child	12-25kg	green	32-02-002-1
2.5	Child	125-150cm	orange	32-02-025-1
3	Adults	< 155cm	yellow	32-02-003-1
4	Adults	155-180cm	red	32-02-004-1
5	Adults	> 180cm	purple	32-02-005-1

**Box of 1**, complete with 1 LT-D, 1 color coded syringe and 1 biteblock

Size	Patient	Patient Size	Connector Color	REF
2	Child	12-25kg	green	32-02-102-1
2.5	Child	125-150cm	orange	32-02-125-1
3	Adults	< 155cm	yellow	32-02-103-1
4	Adults	155-180cm	red	32-02-104-1
5	Adults	> 180cm	purple	32-02-105-1

## Emergency LT-D Set,

Complete with 1 of each LT-D #3, #4, #5, 1 color coded syringe and 1 biteblock

## REF

32-02-209-1

## Instructions for Use

In order to use the LT-D safely the user must first be familiar with these instructions for use.

## Indications for the use of the Laryngeal Tube

The Laryngeal Tube is intended for use in patients for controlled or spontaneous ventilation during anesthesia for procedures that are short in duration, when the patient is considered to have a low risk of aspiration of stomach contents. In addition the Laryngeal Tube is an alternative device to secure the airway during difficult airway management (European Resuscitation Council Guidelines for Resuscitation 2005, Section 4. Adult advanced life support).

## Contraindications for the use of the Laryngeal Tube LT-D in elective situations

Patients who have not fasted, including patients whose fasting cannot be confirmed, and in other situations where there may be retained gastric contents. Situations where gastric contents may be present include, but are not limited to, gross or morbid obesity, pregnancy, multiple or massive injury, acute abdominal or thoracic injury, any condition associated with delayed gastric emptying, or use of opiate medication prior to fasting. Adult patients who are unable to understand instructions or cannot adequately answer questions regarding their medical history, because these patients may be contraindicated for LT-D use.

## Warnings

The user should be familiar with the following warnings when considering or attempting to use the LT-D:

- High airway pressures may divert gas either to the stomach or to the atmosphere.
- After placement, perform standard checks for breath sounds and utilize an appropriate carbon dioxide monitor as required by law or hospital protocol.
- The LT-D is a disposable product and cannot be reused.

## Latex-free

The LT-D is 100% latex-free and should be considered safe to use on patients who are latex sensitive.

## 1. Preparation for use of the Laryngeal Tube LT-D

The product is delivered in sterile condition unless the package is opened or damaged. The LT-D is for single use only and therefore cannot be cleaned or reused.

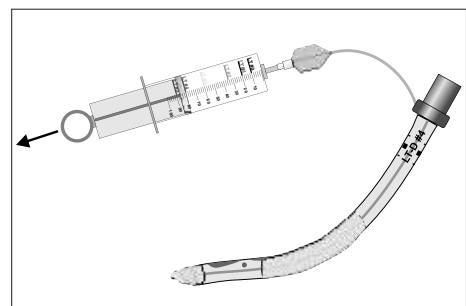
Before the use of the LT-D the device must pass the following performance test:

- Examine the interior of the airway tube to ensure that it is free from blockage or loose particles.
- Inflate the cuffs to check for any possible leakage.

**Caution:** Do not use the LT-D if the cuff deflates.

## 2. Use of the Laryngeal Tube LT-D

- a) Choose the correct LT-D size.
- b) Check that both cuffs are completely deflated and that the LT-D has been lubricated with water-soluble lubricant.
- c) Have a spare LT-D ready and prepared for immediate use
- d) Pre-oxygenation is advisable in all cases

# English

## e) Achieve the appropriate depth of anesthesia

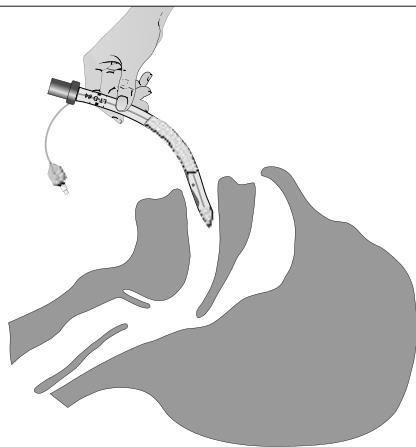
An adequate level of anesthesia is required before the insertion of the LT-D. Standard monitoring techniques should be followed when inducing anesthesia. In general, the depth of anesthesia needed is little more than that required for the insertion of a Guedel-type airway. It is recommended that the less experienced user choose a deeper level of anesthesia.

## f) Positioning the head

The ideal head position for insertion is the “sniffing position”. However the shortness of the tube and the s-shape allows it to be inserted with the head in a neutral position.

## g) Insertion of the Laryngeal Tube LT-D

To position the LT-D it is necessary to use a technique which takes the anatomy into account. Follow the instructions of use in the following sequence:


- The recommended technique is to hold the LT-D like a pen in the area of the black lines (teeth marks).

- With the free hand, hold the mouth open and make sure that the tongue is not folded back during the downward movement of the LT-D Never use force!

- Place the flat edge of the LT-D tip against the hard palate of the patient and slide it along the palate in the midline of the mouth down into the hypopharynx until the center black line is level with the upper teeth.

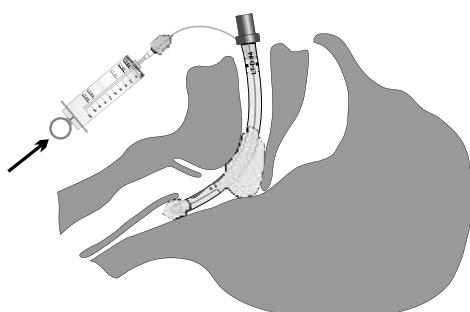
In case of insertion problems a lateral insertion might be useful.


- Inflate both cuffs with the volume which is indicated on the syringe. Due to the specially designed inflation line the proximal cuff is filled first which stabilises the tube. Once the proximal cuff has adjusted to the anatomy of the patient the distal cuff will be automatically inflated.

Consider the recommended volumes:

Size 2	35 ml
Size 2.5	45 ml
Size 3	60 ml
Size 4	80 ml
Size 5	90 ml

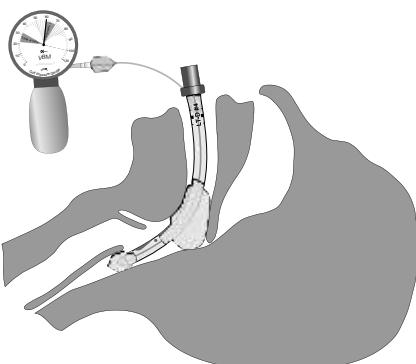- In elective use, especially during spontaneous ventilation, it is recommended to advance the LT-D to the connector. Then inflate the cuffs with the VBM Cuff Pressure Gauge “Universal” to 60cmH<sub>2</sub>O and attach the tube to the breathing circuit. While gently bagging the patient to assess ventilation, simultaneously withdraw the airway until ventilation is easy and free flowing (large tidal volume with minimal airway pressure). Re-adjust cuff pressure and monitor continuously to avoid excessive cuff pressures.

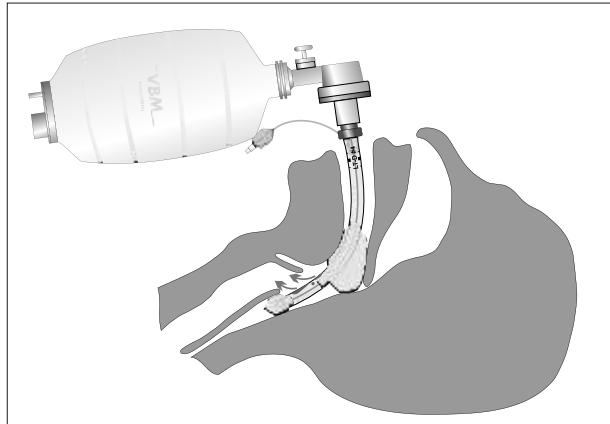- The LT-D should now be properly positioned and the patient can be ventilated.

Check lung ventilation by auscultation, capnography and chest movement. If ventilation is not sufficient, re-position the tube between the teeth marks.

If efficient ventilation is achieved, the LT-D can be fixed in position. The biteblock can do this, but is not required.

## Potential problems with inserting the LT-D

Jaw thrust or chin lift may help to advance the LT-D because it lifts the base of the tongue.

If no sufficient ventilation can be verified, re-position the LT-D between the teeth marks.

It is recommended to insert the LT-D initially deeper. While gently bagging, slowly withdraw the tube with inflated cuffs until ventilation is easy and free flowing (large tidal volume with minimal airway pressure). Withdrawal of the LT-D with inflated cuffs results in a retraction of tissue away from the laryngeal inlet and it prevents the epiglottis or other tissue to be drawn into the ventilation holes of the tube during spontaneous ventilation.

If airway problems persist or ventilation is still inadequate, the LT-D should be removed and an airway established by other means.

An inadequate depth of anesthesia may result in coughing and breath holding during LT-D insertion. Should this occur, the anesthesia should be deepened immediately and manual ventilation resumed.

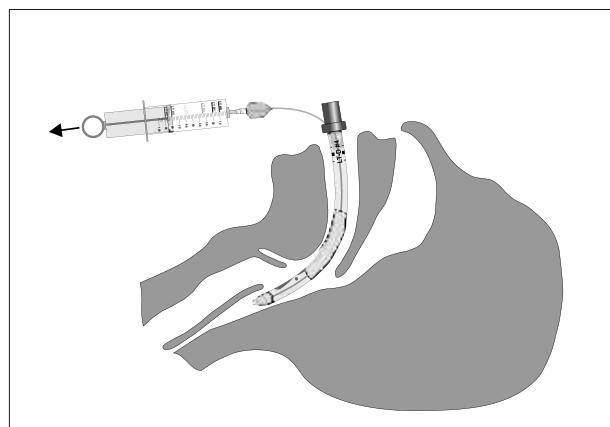
To avoid trauma, force should not be used at any time during insertion.

### h) Removal of the LT-D

Once it is in the correct position, the LT-D is well tolerated until the return of protective reflexes. Intra-cuff pressures should be maintained around 60cmH<sub>2</sub>O.

**Note:** For LT-D removal, it is important that both cuffs are completely deflated with the included syringe to avoid cuff damage.

Removal should always be carried out in an area where suction equipment and the ability for rapid intubation are present.

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